



Eurosafe 60 Disinfectant Washer

COMPLETE & EFFICIENT
THERMAL DISINFECTION

02 9734 0673 | promedco.com.au | contactus@promedco.com.au

FEATURES AT A GLANCE



All in one Automated Cleaning

In a single cycle, Eurosafe 60 pre-washes, cycle-washes, thermally disinfects and dries, so there is no need for manual cleaning. With less time and less effort,

Eurosafe 60 ensures effective and efficient results, eliminating the risk to operators when handling contaminated instruments. With its automatic dosage system to adjust detergent use, it is also environmentally friendly.



Cleans, disinfects and dries safely

The large, 60-litre tank can thermally disinfect wands, loose instruments and containers, allowing for a better use of space thanks to its numerous accessories. Thanks to its user-friendly interface, up to 40 cycles can be programmed and customised by a variety of settings, according to requirements and the equipment being treated, which include washing time and temperature. Eurosafe 60 ensures full and safe traceability: validating and recording all the thermal disinfection cycles operated, saving all data on a USB stick for later transfer to a PC. Eurosafe 60 saves all the information on all the disinfection cycles on a USB stick for later transfer to a computer by an Ethernet connection (available on request) or the data can be instantly accessed by using the optional thermal printer.



Maximum operator safety

Eurosafe 60 has been wholly designed to ensure maximum safety, from its baskets and washing chamber in AISI 316L steel with rounded edges, to its safety system preventing the door from being opened during wash.



Multi-user management

The disinfection process can be managed by several operators (up to 30) with authorised password access. This ensures that every operation is validated, documented operator-linked. Multi-user management
Automatic detergent dispenser



FEATURES



Traceability

The disinfection process is validated and recorded using a USB stick, Ethernet connection, and external printer.



Performance

The large tank, an injection cleaning system and a propelled-air drying system, ensures a faster and more complete cleaning process.



Pro System Connectivity

Eurosafe 60 is even more efficient if connected to other Euronda devices. This allows for complete traceability ensuring quality and safe sterilisation process.



All in one

Pre-washes, washes, and dries in a single cycle, ensuring more effective and efficient results in less time.



User-Friendly

Up to 40 cycles can be programmed and customised thanks to the user-friendly LCD interface.



Accessories for every requirement

The Eurosafe 60 Starter Kit has a wide range of useful accessories, including a full optional range of receptacles, baskets, wand-holders, adaptors and the external thermal printer.



Software

At the end of each cycle the thermal disinfection device generates a file which records all the data on the process, validating the load if the recorded temperatures are compliant. This file can then be read by the management software, thereby all the processes occurring in the sterilisation room can be monitored.



Automatic detergent dispenser

Euroclean 120 and Eurobright 360 detergents are directly injected into the washing chamber by a peristaltic pump controlled by the system. The thermal disinfection autonomously dispenses the correct amount of product thereby reducing waste.



PERFORMANCE

Efficient washing and thermal disinfection

During the wash phase a powerful pump produces strong and continuous multidirection water flow. The device can be adjusted up to a temperature of 93° C effectively disinfecting the instruments.

Forced-air drying

All the thermal disinfection cycles end with a drying phase: a powerful blower, combined with a resistance, injects hot air into the wash chamber from several points to ensure the load is perfectly dry. The hot-air drying system completely eliminates the water deposits both in the external and internal parts of the instrument body. A HEPA filter is optional for absolute safety.

Reduced consumption

Eurosafe 60 has been designed to be totally eco-friendly. Electricity and water consumption have been optimised: 1.7 KW and 12 litres of water for every short cycle. Injection washing system Forced air drying.



USER-FRIENDLY

Easy to use

Following the Euronda philosophy of easy-to-use devices, the Eurosafe 60 control panel provides an easy and rapid access interface.

Customisable

The thermodisinfectior has 20 pre-set cycles with various phases that best fit with wash requirements. In addition, there is the option to set and save 20 other customisable cycles. Creating and changing these cycles is through a password-protection setting.

Inspection ability

The clear tempered double-glass door allows for easy monitoring of the contents during a cycle. Thus, any blockages due to the incorrect positioning of the instruments internally can be detected ensuring that operations are always supervised. Clear tempered double-glass door.

TECHNICAL INFO

Overall dimensions 560 x 825 x 577 mm (L x H x P)

Weight (machine empty with basket and trays) 62 kg

Power supply 230V/~ /50Hz

Total power 2750 W

Pump power 150 W

Noise level < 70 dB(A)

Operating temperature +5°C / +40°C

